


*The subject must not place **ANYTHING** in the mouth for 15 minutes prior to beginning test. This includes non-alcoholic drinks, tobacco products, coffee, breath mints, food, etc.*

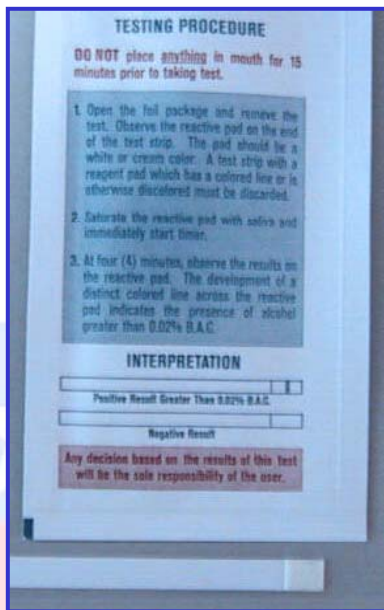

“Negative” Example


✓ Open foil package, remove test, and observe the reactive pad. The pad should be a white or cream color. Discard any tests that have a colored line or are otherwise discolored. The preservative packet is not needed to perform the test and should be discarded.

✓ Saturate the reactive pad with saliva, either from a cup or by applying saliva directly to the pad from the mouth. After 10 seconds, shake off excess saliva.

✓ At four minutes, observe results on the reactive pad. Placing the pad on a white background may make reading the results easier.

✓ Development of a distinct colored line across the reactive pad indicates the presence of alcohol greater than 0.02% Blood Alcohol Concentration (BAC). No line indicates a negative result. Results obtained after more than 5 minutes are invalid.

✓ Positive results must be confirmed within 30 minutes by a Breath Alcohol Machine operated by a certified Breath Alcohol Technician.

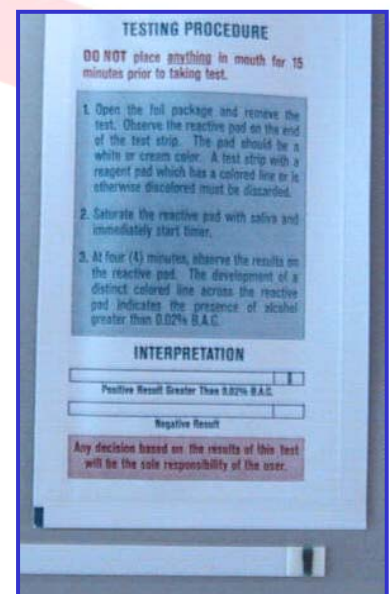

“Positive” Example