




## The Cassette

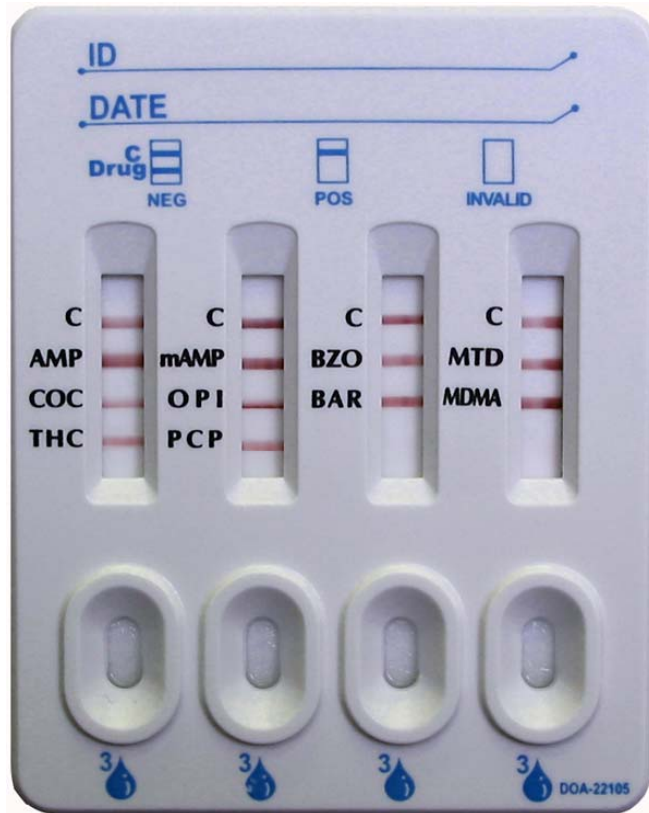

- Screens for 1 - 10 drugs
- Apply 3 drops of urine to each sample well
- Wait 5 minutes to read positive results

# The Cassette Instructions




## STEP 1

Collect specimen and verify temperature is within range (90 –100 F)



## STEP 2


Apply 3 to 4 drops of specimen to each sample well

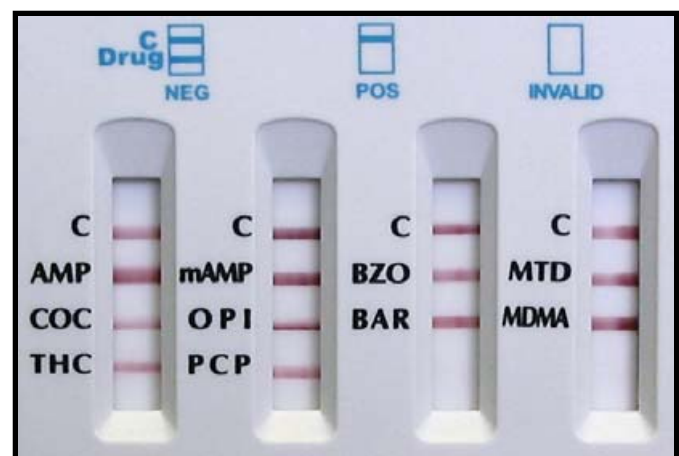

## STEP 3


Read Test Results

# Results Interpretation

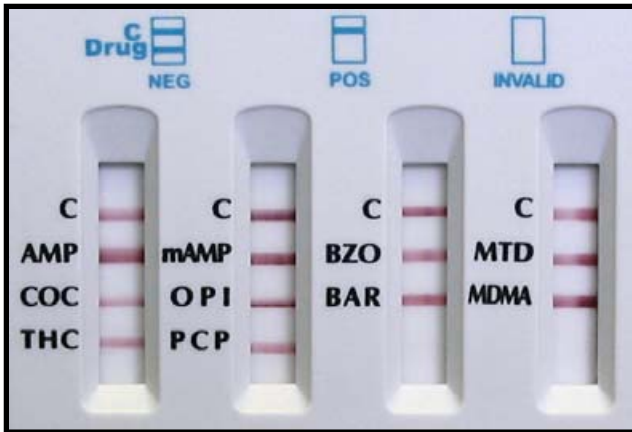


## NEGATIVE

A **Negative** test is indicated by the presence of a test line by each designated drug.

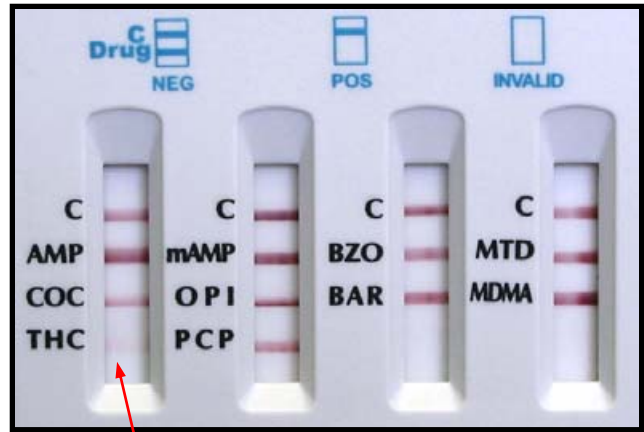


## NEGATIVE

This is also a **Negative** screen. Even a **very light** colored line indicates a negative result.

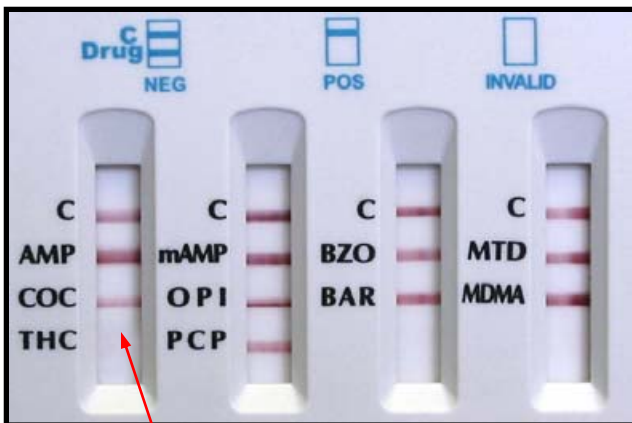

## POSITIVE

A **Positive** test is indicated by the presence of only a "C" Control Line and the **ABSENCE** of a Test Line beside the designated drug.

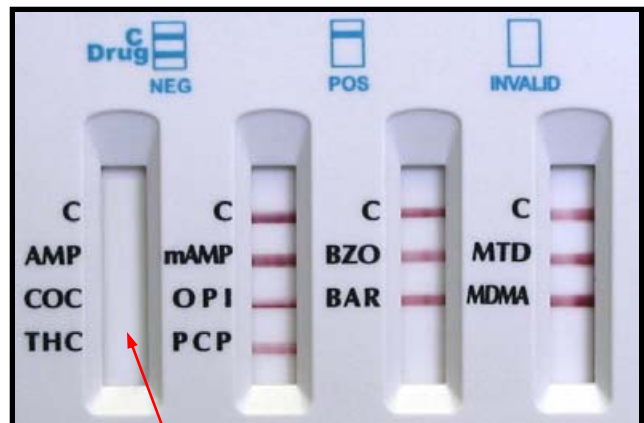

## INVALID

An **Invalid** test is indicated only when the "C" Control Lines and Test Lines are completely missing in one or more of the test windows. In event of an invalid screen a second screen should be run.

*Wait 5 minutes for Positive Results. Do not read results Past 10 Minutes.*

**For Technical Support call: 888-669-4337**